



914752



914750



914753



914755

Re-Order #	Description	Size	Unit
914750	Crash Pad™ - Green	3' x 4'	Each
914752	Crash Pad™ - Blue	5' x 5'	Each
914751	Crash Pad™ with extra nylon cover	3' x 4'	Each
914753	Crash Pad™ with extra nylon cover	5' x 5'	Each
914754	Crash Pad™ with extra cozy cloth cover	3' x 4'	Each
914755	Crash Pad™ with extra cozy cloth cover	5' x 5'	Each

### Purpose

The Crash Pad is a soft-landing zone for kids requiring sensory input or can be used as a giant bean bag for reading & relaxing. Ideal for sensory rooms in schools, clinics, or at home. This pad is filled with soft foam pieces and comes with a durable water resistant anti-bacterial nylon cover.

### Easy to Use

This item ships compressed and will expand to complete fullness within 48 hours. The mesh vents gently release air during use and allow the pad to return to its natural ideal state. To ensure safe crashing, place the pad away from walls and furniture. Do not block mesh vents. The nylon cover easily wipes clean and can be laundered separately. Do not launder the foam pieces.

### Caution

This pad is free of flame-retardant properties and must always be used under the direct supervision of an adult or caregiver. While the zipper is functional, it should remain closed at all times. The internal foam pieces are not intended to be removed, played with, or ingested.

### Easy Care

The nylon cover (without the foam pieces) can be laundered at temperatures below 180°F (82°C). However we do not recommend laundering the inner cover, it should only be wipe cleaned. An optional replacement cover is available and can be easily slipped over the inner cover.