



915325



915330

ReOrder #	Description	Size	Units
915325	Gel-Foam Toilet Seat Cushion	14" x 18"D x 1.5"H	Ea.
915330	Foam Only Toilet Seat Cushion	14" x 18"D x 1.5"H	Ea.
909397	Replacement Straps	One Size	24/pcs

Purpose

The Gel-Foam Toilet Seat Cushion incorporates a top layer of gel and a bottom layer of foam to provide comfort and maximum pressure relief. The baffled gel pack design stabilizes the gel to insure that the gel doesn't migrate away from the high pressure areas. The Foam Toilet Seat Cushion improves the residents comfort and provides pressure relief with a top and bottom layer of foam. The heat-sealed vinyl casing is water proof and cleans easily with disinfectants and is soft and gentle on the skin. There are four hook & loop straps that secure the cushion to the toilet seat.

Easy To Use

Place the Toilet Seat Cushion on the toilet seat with the open end of the horseshoe in the front. Lift the toilet seat and secure the four hook & loop straps (Fig. 1). Each strap is to be threaded through both rectangular rings (Fig. 2) and are closed by placing the end of the strap on top of the opposing side. Pull the strap snug without overtightening.

Fig. 1



Fig. 2



Easy Care

The vinyl seat cushion is heat-sealed and can be cleaned with a mild disinfectant. Each strap can be removed and cleaned by wiping, spraying or soaking in a disinfectant. Do not boil or place in a microwave. If the straps require replacing they can be ordered as (model# 909397). If a tear or rip is observed, the unit should be discarded. Over an extended time the liquid may dissipates.

Caution

Although the gel is non-toxic, medical care should be consulted if the gel is ingested