


Geri-Chair Foot Cradle

Reorder #	Unit
703430	Each

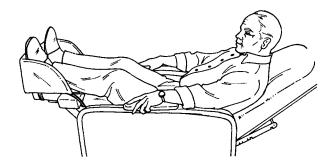

Purpose

The Geri-Chair Foot Cradle is designed to control foot drop and prevent a resident's feet from slipping off the sides of an elevated geri-chair footrest.

Important notice

This product is designed and intended to be used only when the geri-chair is in the reclined position and the footrest is elevated. It must be removed before the geri-chair is placed in the upright position.



- 1. Place the Foot Cradle on the elevated footrest and position the forward strap as shown in illustration A. Secure the buckle and take up as much slack as possible.
- 2. Place the other strap under the bottom part of the footrest as shown in illustration B. Secure the buckle and take up the slack.



The Foot Cradle wipes clean using a damp cloth sponge or sponge and a spray-type cleanser. A mild disinfectant may be used.

