

29 Wells Avenue Yonkers, New York 10701

ADJUSTABLE HEEL-FLOAT WALKER BOOT

| Reorder # | Size | Unit |
|-----------|-----------------------------|---------|
| 503147 | Small | Each |
| 503148 | Medium | Each |
| 503149 | Large/Bariatric | Each |
| 503095 | Replacement Gel Pack(small) | Each |
| 503097 | Replacement Gel Pack(mediur | n) Each |
| 503099 | Replacement Gel Pack(large) | Each |



914-963-2040

Purpose

Skil-Care's Heel-Float Walker Boot floats the heel over an air cavity to prevent pressure sores from forming or aid in the healing of existing sores. It includes a detachable nonslip tread pad that permits limited ambulation and facilitates bed/wheelchair transfer. The foot support includes adjustable straps that permit accommodation of a wide range flexion contracture needs. A firm plastic insert provides additional support. The gel insert beneath the ankle relieves pressure and cools the skin.

Application

The Adjustable Heel-Float Walker Boot may be used for either the right or left foot. Select the proper size for your patient.

Bed

- 1. Remove the detachable non-slip tread pad (Illustration B). For infection control purposes, the tread pad must be removed before patient's foot is placed on the bed.
- 2. Place the patient's lower extremity in the boot. Make certain that the heel is suspended over the air cavity (Illustration A).
- 3. Secure the Velcro^R ankle and toe straps as shown in Illustration A.

4. Adjust the two foot support straps to acommodate your patient's plantar flexion (Illustration C).

Ambulation and Transfer

- 1. Seat patient on side of bed.
- 2. Attach non-slip tread pad (Illustration C)
- 3. Ambulate or transfer patient according to your facility's policies and procedures.
- 4. Remove tread pad before placing patient's foot on bed.

Important

Remove this device every 8 to 12 hours and check for ischemia or discoloration.


Easy Care

The Adjustable Heel-Float Walker Boot is launderable and may be washed and dried at temperatures that do not exceed 180°F (82°C). Do not use chlorine bleach. For infection control purposes, the outer nylon cover should be cleaned using a spray-and-wipe cleanser and disinfected with a non-chlorine disinfectant when patient is returned to bed. **Remove gel insert (Illustration D) and plastic foot support insert (Illustration D) before laundering.**

Guarantee

The Adjustable Heel-Float Walker Boot is guaranteed against defects in materials and workmanship under conditions of normal use for six months following date of purchase.



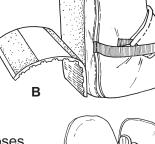







Advanced design for nursing and therapy.™