


SLIDE-GUARD CUSHION

Reorder #	Description	Foundation	CoverFabric
□751210	Leveled Gel Cushion	Firm	Vinyl & LS I
□751230	Leveled Gel Cushion	Soft	Vinyl & LS I
□751260	Leveled Foam Cushion	Soft	Vinyl & LS I
□751265	Leveled Foam Cushion	Soft	Vinyl
□751269	Leveled Foam Cushion	Soft	Vinyl
□751280	Leveled Foam Cushion	Firm	Vinyl & LS I
□751316	Wedged Gel Cushion	Soft	Vinyl & LS I
□751310	Wedged Gel Cushion	Firm	Vinyl & LS I
□751370	Wedged Foam Cushion	Soft	Vinyl & LS I
□751320	Wedged Gel Cushion	Soft	Vinyl & LS I
□751324	Wedged Gel Cushion	Soft	Vinyl & LS I
□751314	Wedged Gel Cushion	Firm	Vinyl & LS I
□751350	Wedged Foam Cushion	Firm	Vinyl & LS I

PURPOSE

Skil-Care's SLIDE-GUARD CUSHION helps to position patients who slide down in the wheelchair. In addition, the cushion's pommel acts as a leg abductor. Each Slide-Guard Cushion has ties for the purpose of securing the cushion to the wheelchair. For patient safety, these ties must be fastened to the chair.

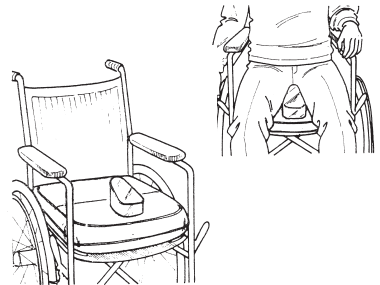

COVER LAUNDERING AND CUSHION CARE

Both the polyester and sheepskin covers remove for laundering. The covers may be washed and dried at temperatures up to 180°F (182°C). Bleaching is not recommended. The vinyl cushion may be cleaned with a mild spray-type cleaner and damp cloth or sponge.